

Laboratory Test Updates

(effective on November 04, 2024)

| | Test Code | Test Name | Name Change | Methodology | Performed/Reported Schedule | Specimen Requirements | Reference Interval | New Tests | Additional Information | Internalized Tests | Reactivated Tests | Inactive |
|---|-----------|-----------------------------------|-------------|-------------|-----------------------------|-----------------------|--------------------|-----------|------------------------|--------------------|-------------------|----------|
| 1 | C8623524 | Myositis Panel | | X | | | | | | | | |
| 2 | C8925665 | Thrombophilia Panel-1 | | | | | | X | | | | |
| 3 | C8925664 | Multiple Myeloma Panel | | | | | | X | | | | |
| 4 | C8237401 | Bicarbonate | | | | X | | | | | | |
| 5 | C8459102 | Vitamin B3 (Niacin, Nicotinamide) | | | | X | | | | | | |
| 6 | C8308002 | Hexosaminidase, Total | X | | | | | | | | | |

| | | | |
|---|--|----------------|----------|
| 1 | Myositis Panel Test Code : 8623524 - Laboratory Services | | |
| | Method : IMMUNOBLOTTING - Performed : Wed 08:00 - Reported : Thu 18:00 | | |
| | Sample Name | Container | Amount |
| | Serum | Separator Tube | 1mL |
| | | | Transfer |
| | | | 2-8°C |

2

Thrombophilia Panel-1

Test Code : **8925665** - Laboratory Services

Method : - Performed : **Mon 08:00 Wed 08:00 Fri 08:00** - Reported : **Tue 18:00 Thu 18:00 Mon 18:00**

| Sample Name | Container | Amount | Transfer |
|--------------------------|-----------------------|--------|----------|
| | Slide | | 18-24°C |
| CSF | Slide | | 18-24°C |
| Biologic Fluid | Slide | | 18-24°C |
| Bone Marrow | Slide | | 18-24°C |
| Serum | Separator Tube | | 2-8°C |
| Plasma (Citrate), Frozen | Blue (Sodium citrate) | | <0°C |

Additional Information

This panel includes; Blood Smear and Bone Marrow Aspirate Preparation Staining, Anti-Beta-2 Glycoprotein 1 Antibody, IgM, Anti-Beta-2 Glycoprotein 1 Antibody, IgG, Lupus Anticoagulant (LA), Anti-Phospholipid Antibody, IgM, Anti-Phospholipid Antibody, IgG, Activated Protein-C Resistance (APC), Antithrombin III, Protein S Activity, Protein C Activity. All sample types defined in the test panel are required.

3

Multiple Myeloma Panel

Test Code : **8925664** - Laboratory Services

Method : - Performed : **Mon-Sat 08:00** - Reported : **5 days**

| Sample Name | Container | Amount | Transfer |
|-------------------------|--|--------|----------|
| Serum and 24-hour Urine | Separator Tube + Plastic 24 Hour Urine Container | | 2-8°C |

Additional Information


This panel includes; Lambda Light Chain, Free, Serum, Kappa Light Chain, Free, Serum, Kappa Light Chain, Serum, Lambda Light Chain, Serum, IgM, IgA, IgG, Immunofixation Electrophoresis (IFE), Serum, Qualitative, Immunofixation Electrophoresis (IFE), Urine (24 hours), Quantitative Free Light Chains, Beta-2-Microglobulin, Serum

4

Bicarbonate
Test Code : **8237401** - Laboratory Services

Method : **PHOTOMETRY** - Performed : **Mon-Sat 12:00** - Reported : **Mon-Sat 18:00**

| Sample Name | Container | Amount | Transfer |
|-------------|------------------------------|--------|----------|
| Serum | Separator Tube (Bicarbonate) | 0,5mL | 2-8°C |

Additional Information 


Serum
It is recommended to take the blood sample into a gel tube, centrifuge within 2 hours, and transfer it without opening the tube cap after centrifugation.

5

Vitamin B3 (Niacin, Nicotinamide)
Test Code : **8459102** - Laboratory Services

Methodology : **LC-MS/MS** - Performed : **Wed 08:00** - Reported : **1 week**

| Specimen Type | Container | Volume | Transport |
|---------------|----------------|--------|-----------|
| Serum, Frozen | Separator Tube | 2mL | <0°C |

Additional Information 

Protect sample from light during collection, storage, and shipment.

6

Hexosaminidase, Total
Test Code : **8308002** - Laboratory Services

Methodology : **FLUOROMETRY** - Performed : **Mon 08:00** - Reported : **Fri 18:00**

| Specimen Type | Container | Volume | Transport |
|--------------------|-----------------|--------|-----------|
| Whole Blood (EDTA) | Lavender (EDTA) | 10mL | 2-8°C |
| Whole Blood (ACD) | ACD-Yellow Top | 8mL | 2-8°C |

